

## BioCLIA Autoimmune Calibrator Set, SS-B/La

Cat.No.	Kit Size
MY00201	2 X 1 mL
MY00252	4 X 1 mL

### INTENDED USE

The BioCLIA Autoimmune Calibrator Set, SS-B/La is intended for the calibration of the BioCLIA SS-B/La performed on the BioCLIA 6500 and BioCLIA 500.

For professional *in vitro* diagnostic use only.

### SUMMARY AND EXPLANATION

Anti-nuclear antibodies (ANAs) are a class of auto-antibodies with different binding affinities specific to different nuclear antigens. Generally, ANAs include extractable nuclear antigen (ENA) antibodies and un-extractable nuclear antigen antibody.<sup>1</sup> Determination of ANAs has significant correlation to the diagnosis of Sharp Syndrome, Systemic Lupus Erythematosus (SLE), Sjögren's Syndrome, progressive systemic sclerosis, polymyositis/dermatomyositis, overlap syndrome, and limited types of progressive systemic sclerosis (CREST syndrome). Anti-SS-B antibodies, also known as anti-La antibodies, are commonly found in primary Sjögren's Syndrome, an autoimmune disorder that affects the exocrine glands. The presence of anti-SS-B and anti-SS-A (anti-Ro antibodies) are found in 30-60% of Sjögren's Syndrome patients, but anti-SS-B antibodies are rarely found alone.<sup>2,3</sup>

Anti-SS-B antibodies are also found in SLE; however, Sjögren's Syndrome is normally also present.<sup>4</sup> In Sjögren's Syndrome, anti-SS-B antibodies correlate with early onset, increased disease duration, parotid gland enlargement, disease outside the glands and infiltration of glands by lymphocytes.<sup>5</sup> The La antigen is a 48 kDa transcription termination factor of RNA polymerase III, which associates with the Ro-RNP complex.<sup>6,7,8,9</sup>

The La antigens are expressed on the surface of cells undergoing apoptosis and may cause the inflammation within the salivary gland by interaction with cells of the immune system. The antibodies may also be produced through molecular mimicry, where cross reactive antibodies bind to both virus and human proteins. This may occur with one of the La antigens and may subsequently produce antibodies to other proteins through a process known as epitope spreading. The retroviral gag protein shows similarity to the La protein and is proposed as a possible example for molecular mimicry in Sjögren's Syndrome.<sup>5,10</sup>

### MATERIALS SUPPLIED

- SS-B/La Calibrator 1** Barcode labeled tubes with buffer containing human antibodies to SS-B/La in stabilizers and preservatives. Ready to use, 1 mL.

CAL	1
-----	---

Preservatives: 0.0015% < Proclin 300 < 0.6%.

- SS-B/La Calibrator 2** Barcode labeled tubes with buffer

containing human antibodies to SS-B/La in stabilizers and preservatives. Ready to use, 1 mL.

CAL	2
-----	---

Preservatives: 0.0015% < Proclin 300 < 0.6%.

The Calibrator Code contains calibrators' information is provided in each kit.

### WARNINGS AND PRECAUTIONS

- For professional *in vitro* diagnostic use only.
- Do not use any calibrators beyond their expiration dates.
- Do not mix calibrators from different lots unless specified.
- Instructions must be carefully followed for using and storing of calibrators. Any modification in procedure may interfere with the results.
- Calibrators and contaminated vials must be handled strictly following safety guidelines or rules of biological hazards to ensure the users' and environmental safety.
- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established!

### Precautions:



Human serum is added in the calibrators.

- The human derived material in this product was tested by FDA approved methods and found nonreactive for Hepatitis B Surface Antigen (HBsAg), Anti-HCV and HIV 1/2 antibodies. Handle as if potentially infectious.<sup>7</sup> Avoid contacting with skin and eyes. Do not empty into drains. Wear suitable protective gloves and clothing.



- Proclin 300 is added in the calibrators at concentration between 0.0015% - 0.6%.
- Calibrators contain chemical and biological components. Avoid ingesting or splashing onto skin and mucous membrane. If direct contact with calibrators happens, rinse immediately the contact surface with plenty of water and see a doctor if necessary.

### STORAGE CONDITIONS

- Store the kit at 2-8 °C.
- The shelf life of the unopened kit is 12 months from day of production.

- Vial opened calibrators could be used for 28 successive days, exposure no more than 2 hours each time when kept uncapped and is good for up to 20 calibrations, after which the reagent must be discarded.
- Avoid repeated freezing and thawing.

## ASSAY PROCEDURE

Detailed information about operating the BioCLIA instruments can be taken from the Instrument User's Manual.

Note that, it is important to perform all routine maintenance procedures for optimal performance.

### Assay Calibration

The BioCLIA Autoimmune Reagent Kit utilizes a predefined lot specific Master Curve which is uploaded into the instrument via the barcode provided in the main reagent kit. The Calibrator Code contains calibrator information is then scanned. Based on the results of running two calibrators, the instrument specific Working Calibration Curve is generated and is used to calculate the concentration from the RLU obtained for each patient.

For each new lot of reagent, please calibrate prior to the first time use, and every 28 days thereafter. The software will not allow the lot to be used if the above requirements are not meet.

### Programming and Running samples

1. Put the kit into the corresponding position of the reagent chamber of the fully automatic chemiluminescence analyzer. The information of the kit can be uploaded into the instrument system through the scanning of reagent barcode, and can also be set through the supporting software of the instrument.
2. The information of calibrator / quality control is identified by scanning the calibrator / control barcodes, and the position of calibrator / quality control is assigned in the instrument system.
3. The sample to be tested is placed on the instrument sample rack chamber, and the corresponding test information is edited through the instrument supporting software.
4. Start the operation procedure, and all calibrator / quality control / sample processing steps will be automatically executed.

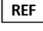

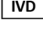
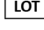
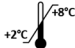


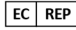




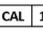
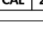
## TRACEABILITY

The reported values were determined over multiple runs on the BioCLIA 6500 and BioCLIA 500 using specific lots of reagents against an in-house standard. DGP IgA results are reported in RU/mL which is interpreted from relative light unit (RLU). Method comparison test showed good sensitivity and specificity of tested assay.

## LIMITATIONS

- The calibrators are designed for calibration of the same lot of BioCLIA Autoimmune Reagent Kit.
- The calibrators can be kept uncapped onboard the instrument up to 2 hours for each time of usage. And a total up to 20 calibrations are suggested, for any longer period of time, the reagent should be discarded, otherwise may result in improper calibration of the assay and which can give improper results.

## SYMBOLS

	Catalog Number		Use-by date
	In Vitro diagnostic medical device		Lot Number
	Store between +2°C and +8°C		Consult Instruction for Use
	Manufacturer		Authorized Representative in the European Community
	CE Marking		Contains Sufficient for <n> Tests
	Biological Risk		GHS07 Warning
	Calibrator 1		
	Calibrator 2		

## REFERENCE

1. Kumar Y, Bhatia A, Minz RW. Antinuclear antibodies and their detection methods in diagnosis of connective tissue diseases: a journey revisited. *Diagnostic pathology* 2009;4:1.
2. Hernández-Molina G, Leal-Alegre G, Michel-Peregrina M. The meaning of anti-Ro and anti-La antibodies in primary Sjögren's syndrome. *Autoimmunity reviews* 2011;10:123-25.
3. Kassan SS, Moutsopoulos HM. Clinical manifestations and early diagnosis of Sjögren syndrome. *Archives of internal medicine* 2004;164:1275-84.
4. Yamamoto K. Pathogenesis of Sjögren's syndrome. *Autoimmunity reviews* 2003;2:13-18.
5. Wenzel J, Gerdson R, Uerlich M, Bauer R, Bieber T, Boehm I. Antibodies targeting extractable nuclear antigens: historical development and current knowledge. *British Journal of Dermatology* 2001;145:859-67.
6. Hernández-Molina G, Leal-Alegre G, Michel-Peregrina M. The meaning of anti-Ro and anti-La antibodies in primary Sjögren's syndrome. *Autoimmunity reviews* 2011;10:123-25.
7. Deshmukh US, Bagavant H, Lewis J, Gaskin F, Fu SM. Epitope spreading within lupus-associated ribonucleoprotein antigens. *Clinical Immunology* 2005;117:112-20.
8. Ben-Chetrit E. The molecular basis of the SSA/Ro antigens and the clinical significance of their autoantibodies. *Rheumatology* 1993;32:396-402.
9. Venables PJ. Sjögren's syndrome. *Best Practice & Research Clinical Rheumatology* 2004;18:313-29.
10. Richmond JY, McKinney RW. Biosafety in microbiological and biomedical laboratories: U.S.GPO. 1999.



**HOB Biotech Group Corp., Ltd.**  
C6 Building, No. 218 Xinghu Road, Suzhou Industrial Park,  
Suzhou, Jiangsu, 215123, China

## CONTACT INFORMATION:

TEL (+86)512-69561996

Fax (+86)512-62956652

WEBSITE: [www.hob-biotech.com](http://www.hob-biotech.com)

CUSTOMER SERVICE: HOB Biotech Group Co., Ltd

CUSTOMER SERVICE: TEL (+86)4008601202



**EUROPE REPRESENTATIVE:** Emergo Europe

**ADDRESS/LOCATION :**

Prinsessegracht 20, 2514 AP The Hague, The Netherlands



**The eIFU is available on Website:**

<http://en.hob-biotech.com/usercenter/login.aspx>

**TECHNICAL ASSISTANCE**

For technical assistance, contact your National Distributor.

Date of issue: 17<sup>th</sup> March 2019

Date of revision: 20<sup>th</sup> December 2021

Change Control Number: CN21129E

Version: A/1 (EN)