


Prealbumin/Transthyretin

Control Slides

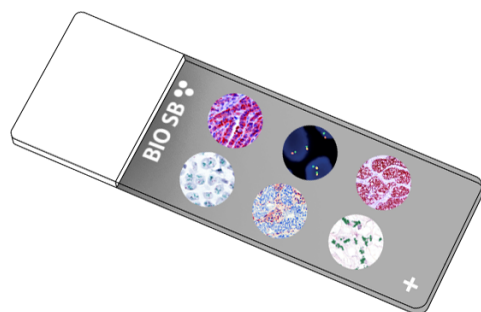

Intended Use

For In Vitro Diagnostic Use.

Summary and Explanation

TTR (TTR, transports thyroxine and retinol, or TBPA) encodes one of the three prealbumins including alpha-1-antitrypsin, transthyretin and orosomucoid. Transthyretin is a carrier protein; it transports thyroid hormones in the plasma and cerebrospinal fluid, and also transports retinol (vitamin A) in the plasma. The liver secretes Transthyretin into the blood, and the choroid plexus secretes TTR into the cerebrospinal fluid.

TTR mutations are associated with amyloid deposition, predominantly affecting peripheral nerves or the heart. TTR misfolding and aggregation is known to be associated with the Amyloid Diseases, Senile Systemic Amyloidosis, Familial Amyloid Polyneuropathy, and Familial Amyloid Cardiomyopathy. Transthyretin amyloidosis is a slowly progressive condition characterized by the buildup of abnormal deposits of amyloid (Amyloidosis) in the body's organs and tissues.

It has been reported that TTR can be used as an immunohistochemical marker for choroid plexus papillomas, as well as carcinomas. The TTR gene has been found to be suppressed in Hepatic Carcinoma, where the TTR gene was found to be defective in its gene structure, which may have a relevance in its pathogenesis.

Presentation

Five slides of Prealbumin/Transthyretin positive tissues, each mounted on Hydrophilic Plus Slides, provided in a plastic mailer.

<i>Catalog No.</i>	<i>Quantity</i>
BSB-9352-CS	5 slides
BSB 3622	5 slides

Storage Store at 20-25°C

Precautions

1. For professional users only. Results should be interpreted by a qualified medical professional.
2. Ensure proper handling procedures are used with this reagent.
3. Always wear personal protective equipment such as a laboratory coat, goggles, and gloves when handling reagents.
4. Dispose of unused solution with copious amounts of water.
5. Follow safety precautions of the heating device used for epitope retrieval (TintoRetriever Pressure Cooker or similar).
8. For additional safety information, refer to Safety Data Sheet for this product.
9. For complete recommendations for handling biological specimens, please refer to the CDC document, "Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories" (see References in this document).

Stability

This product is stable up to the expiration date on the product label.

Do not use after expiration date listed on the package label.

IHC Protocol

1. Subject tissues to heat induced epitope retrieval (HIER) using a suitable retrieval solution such as ImmunoDNA Retriever with Citrate (BSB 0020-BSB 0023) or EDTA (BSB 0030-BSB 0033).

2. Any of three heating methods may be used:

a. TintoRetriever Pressure Cooker or Equivalent

Place tissues/slides in a staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA and place on trivet in the pressure cooker. Add 1-2 inches of distilled water to the pressure cooker and turn heat to high. Incubate for 15 minutes. Open and immediately transfer slides to room temperature.

b. TintoRetriever PT Module or Water Bath Method

Place tissues/slides in a pre-warmed staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA at 95°-99° C. Incubate for 30-60 minutes.

c. Conventional Steamer Method

Place tissues/slides in a pre-warmed staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA in a steamer, cover and steam for 30-60 minutes.

3. After heat treatment, transfer slides in ImmunoDNA Retriever with Citrate or EDTA to room temperature and let stand for 15-20 minutes.
4. For manual staining, perform antibody incubation at ambient temperature. For automated staining methods, perform antibody incubation according to instrument manufacturer's instructions.
5. Wash slides with ImmunoDNA washer or DI water.
6. Continue IHC staining protocol. Wash slides between each step with ImmunoDNA washer solution.

Abbreviated Immunohistochemical Protocol

Step	ImmunoDetector AP/HRP	PolyDetector AP/HRP	PolyDetector Plus HRP
Peroxidase/AP Blocker	5 min.	5 min.	5 min.
Primary Antibody	30-60 min.	30-60 min.	30-60 min.
1st Step Detection	10 min.	30-45 min.	15 min.
2nd Step Detection	10 min.	Not Applicable	15 min.
Substrate- Chromogen	5-10 min.	5-10 min.	5-10 min.
Counterstain / Coverslip	Varies	Varies	Varies

Abbreviated IF Protocol

Step	Incubation Time
Rinse slides in IF wash buffer	5 minutes
Drain and wipe excess IF wash buffer off slide	
Conduct remaining steps in the dark	
Apply Antibody	30-60 minutes
Rinse with 3 changes of IF wash buffer	3x15 minutes each
Coverslip with IF mounting medium	

Mounting Protocols

For detailed instructions using biodegradable permanent mounting media such as XyGreen PermaMunter (BSB 0169-0174) or organic solvent based resin such as PermaMunter (BSB 0094-0097), refer to PI0174 or PI0097.

Product Limitations

Due to inherent variability present in immunohistochemical procedures (including fixation time of tissues, dilution factor of antibody, retrieval method utilized, and incubation time), optimal performance should be established through the use of positive and negative controls. Results should be interpreted by a qualified medical professional.

References

1. Razavi H, et al. "Benzoxazoles as transthyretin amyloid fibril inhibitors: synthesis, evaluation, and mechanism of action". Angew. Chem. Int. Ed. Engl. 2003; 42 (24): 2758-61.
2. Zeldenrust SR, Benson MD. "Familial and senile amyloidosis caused by transthyretin". In Ramirez-Alvarado M, Kelly JW, Dobson C. Protein misfolding diseases: current and emerging principles and therapies. New York: Wiley. 2010; pp. 795-815.
3. Westermarck P, et al. "Fibril in senile systemic amyloidosis is derived from normal transthyretin". Proc. Natl. Acad. Sci. U.S.A. 1990; 87 (7): 2843-5.
4. Holmgren G, et al. "Clinical improvement and amyloid regression after liver transplantation in hereditary transthyretin amyloidosis". Lancet. 1993; 341 (8853): 1113-6.
5. TTR transthyretin (Homo sapiens)
<https://www.ncbi.nlm.nih.gov/gene/7276>
6. Li X, Buxbaum JN. "Transthyretin and the brain re-visited: is neuronal synthesis of transthyretin protective in Alzheimer's disease?". Mol Neurodegener. 2011; 6 (1): 79.
7. Gu JR, et al. Transthyretin (prealbumin) gene in human primary hepatic cancer. 1991; europepmc.org/abstract/MED/1666289
8. U.S. Department of Health and Human Services: Centers for Disease Control and Prevention. Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories. Supplement / Vol. 61, January 6, 2012.
<https://www.cdc.gov/mmwr/pdf/other/su6101.pdf>

Symbol Key / Légende des symboles/Erläuterung der Symbole

EC REP QAdvis EAR AB Ideon Science Park Scheelevägen 17 SE-223 70 Lund, Sweden	 Storage Temperature Limites de température Zulässiger Temperaturbereich	 Manufacturer Fabricant Hersteller	REF Catalog Number Référence du catalogue Bestellnummer
IVD In Vitro Diagnostic Medical Device Dispositif médical de diagnostic in vitro In-Vitro-Diagnostikum	 Read Instructions for Use Consulter les instructions d'utilisation Gebrauchsanweisung beachten	 Expiration Date Utiliser jusque Verwendbar bis	LOT Lot Number Code du lot Chargenbezeichnung