

Specific and sensitive diagnosis Mycoplasma pneumoniae IgG-, IgA-, IgM-ELISA





Definition/systematics



ycoplasma pneumoniae belongs to the family Mycoplasmataceae. They constitute together with four other families the self-contained class mollicutes (soft-skinned).

Mycplasmae don't possess a cell wall. They are enclosed by a membrane which is very flexible thus allowing extreme changes of their shape (pleomorphy). Some species, e.g., *Mycoplasma pneumoniae*, possess a structure which resembles to a cytosceleton and which stabilizes the cell shape. With a mean size of 300-800 nm, mycoplasmae represent the smallest bacteria which are capable of autonomous replication. Like viruses they are able to penetrate membrane filters.

Mycoplasmae possess a very small genome (580-2200 kbp) which strongly constricts their synthesising capability.

To date, more than 100 species of the family mycoplasmataceae are known. The majority of them have been found exclusively in animals. In humans, in the meantime, 14 species have been isolated which colonise the mucous epithelia of the respiratory and urogenital tract. Only few of these species are pathogenic.

Mycoplasma pneumoniae belongs to the human-pathogenic species. The agent occurs in two forms which differ in regard to the P1 adhesin.

M. pneumoniae is an obligate extracellular parasite of the mucous epithelia of the respiratory tract which is characterised by high host specificity.

Infection



The infectivity of *M. pneumoniae* is minor. Transmission from human to human takes place by droplet infection.

Firstly, *M. pneumoniae* colonises the mucous epithelia of the respiratory tract. The following virulence factors are decisive: adherence to the epithelia cells, mobility, adaptation to the host, and induction of pathological immune responses. Mycoplasmae adhere to the cilia of the ciliated epithelium. Already few hours later ciliostasis occurs with subsequent destruction of the epithelium. Adherence and colonisation are a pre-condition for further spread of the pathogen in the respiratory tract.

Besides the common respiratory tract infections, *Mycoplasma pneumoniae* may also be responsible for systemic spread. The pathological mechanism which may be responsible for overcoming the epithelial barrier and for entrance of the pathogen into the blood circulation, to date, is not yet clear. Probably the adhesive characteristics of the agent change.

Clinical manifestation



pneumoniae causes diseases in both the upper and the lower respiratory tract. Besides tracheo-bronchitis and atypical pneumonia, pharyngitis, laryngitis, otitis media, and myringitis may be induced. Meningitis, meningo-encephalitis, polyneuritis (Landry-Guillain-Barré syndrome), myocarditis, pericarditis, myalgia, arthralgia, arthritis, hemolytic anemia, thrombocytopenic purpura as well as affections of the skin (e.g., Stevens-Johnson syndrome) are known extrapulmonary complications and sequelae, respectively.

Epidemiology



• pneumoniae pneumoniae is of worldwide distribution. The pathogen may cause both endemics and epidemics during any season. Epidemics caused by M. pneumoniae have been observed every three to seven years.

Infections

Course of disease/therapy

Incubation time after *M. pneumoniae* contamination is between 10 and 21 days. *M. pneumoniae* infections may run both an asymptomatic and symptomatic course. However, the symptoms including fever, dry cough, and headache are very unspecific.



In contrast to infections in adolescents and adults, infections in small children below 5 years of age show a milder course. In small children the infection often is self-limiting.

In general, primary infections seem to take a more favourable course than reinfections. *M. pneumoniae* reinfections normally provoke an inflammatory response which is much stronger than that in primary infections because of auto-immune processes.

After an expired infection persistent pathogens are frequently detectable for weeks or months. The so-called switching, i.e., the spontaneous switching on and switching off of membrane protein synthesis, enables short-dated changes of the membrane's antigenic characteristics. Thus, the pathogen may escape from the present immune response.

The classic therapy includes tetracyclines. Macrolides and chinolones are also used. Because of the lack of a cell wall, cell wall-specific antibiotics are ineffective.

General incidence/prevalence

pneumoniae infections belong to the most frequent causes of community acquired tracheobronchitis and pneumonia in children and adolescents. The highest incidence is between the 5th and 15th year of age. Up to 20% of all community acquired respiratory diseases may be ascribed to *M. pneumoniae*. In 5-10% of these diseases atypical pneumonia may develop.



The antibody prevalence is age-dependent as determined in a healthy population. In small children the IgG seroprevalence is about 15%, in adults it may reach 40-50%. IgA antibodies are not detectable in small children. In a healthy population of adults up to 30% IgA antibodies have been described.

Diagnosis

As the clinical symptomatology of a *M. pneumoniae* infection does not show pathogen-specific characteristics, the diagnostic differentiation from other pathogens such as viruses and gram-positive bacteria is decisive for appropriate therapy.



The diagnosis can be based on direct detection and serology. Detection of the pathogen has been regarded as efficient diagnosis at acute, early stages of disease.


Isolation of the pathogen by culture has been considered the reference method. However, it is too insensitive and time-consuming (6-14 days). A good quality pathogen DNA detection system (PCR) is not yet commercially available.


To date, serology has been considered the method of choice for diagnosis of *M. pneumoniae* infections. The complement fixation test (CFT) represents the classic antibody detection. The CFT cannot discriminate between antibody isotypes. Agglutination tests cannot discriminate between antibody classes either. Both test systems detect mainly the IgM antibody response. In reinfections both CFT and agglutination tests provide predominantly negative results. Nevertheless these test systems currently dominate the ELISA technology. By using ELISA, IgG, IgA, and IgM, antibodies can be differentiated. The crucial factor for a specific and sensitive ELISA is the antigen.


Antigenic structure



The adherence of *M. pneumoniae* to the host cell has been regarded as the decisive virulence factor. The docking of the pathogen is effected via the terminal or tip structure which is characteristic for the agent (see figures).

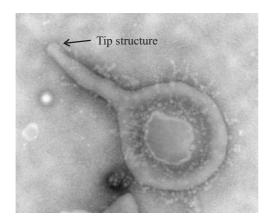

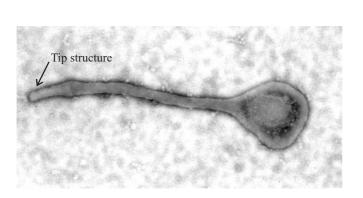

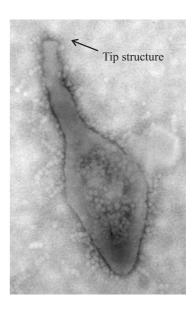

Figures: Electron microscopic pictures of *M. pneumoniae*. By kind permission of Prof. G. Christiansen, Dr. S. Birkelund, University of Aarhus, Denmark, 2004.

The important highly immunogenic proteins in principal are distributed all over the surface of the pathogen. On the terminal structure they may also occur as larger clusters. Out of the more than 10 known immunogenic proteins, the P1 (170 kDa) protein is of predominant importance.

Taking these facts into consideration, the antigens for the **Mycoplasma pneumoniae ELISAs medac** have been selected.

The antigens of the **Mycoplasma pneumoniae-IgG-** and **IgA-ELISA medac** consist of a mixture of various recombinant proteins.

The Mycoplasma pneumoniae-IgM-ELISA medac is based on native, purified antigen.

Quantification

The possibility of quantification with Mycoplasma pneumoniae-IgG- and IgA-ELISA medac provides a good prerequisite for monitoring proven infections whilst in regard to IgM a yes/no result is considered sufficient.



The single-point quantification represents a user-friendly, economic method for ascertainment of quantitative results. The medac single-point quantification is easy to perfom, easily programmable, and the software-based results are rapidly interpretable.

Formula for calculation of concentration

Concentration [AU/ml] =
$$b/\left(\frac{a}{OD_{corrected}}-1\right)$$


AU = Arbitrary Unit


The curve parameters a and b have to be taken from the batch-specific data sheet.

$$OD_{corrected} = \frac{OD \text{ set value of the calibrator}}{OD \text{ measured value of the calibrator}} \quad \text{x} \quad OD_{measured}$$

Determination of concentration via standard curve and calibrator (single-point quantification)

The medac single-point quantification is a highly precise method which provides results that are equivalent to a conventional standard curve (see graphs). The measuring range spans from 9 to 130 AU/ml.

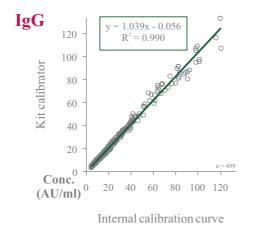

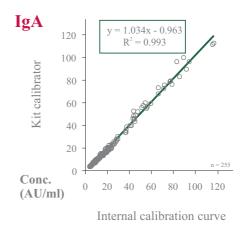

The cut-off is located at 10 AU/ml. The grey zone spans from 9 to 11 AU/ml. Samples with values below the grey zone (< 9 AU/ml) are assessed negative and values beyond the grey zone (> 11 AU/ml) are assessed positive.

Sensitivity/specificity



For determination of sensitivity and specificity a reference system is needed. However, in regard to mycoplasma serology no "Gold Standard" exists. This results in the difficulty to determine these parameters. To obtain data in spite of lack of reference tests, a reference system was defined (Reference I) consisting of a particle agglutination test (PAT) and a commercialised ELISA (C-EIA). For determination of sensitivity* and specificity of the medac ELISAs, sera were selected that had provided concordant positive and negative results, respectively, with both PAT and C-EIA. These sera were measured with Mycoplasma pneumoniae-IgG, IgA-, and IgM-ELISA medac (MP ELISA). All sera with "false negative" results in the medac tests were retested with another commercialised ELISA (Reference II). Considering these results, sensitivity was recalculated (corrected sensitivity).

	IgG	Refe		
_		-	+	
LISA	-	32	6	
MP EI	+	7	38	
Σ				83

Specificity	82%
Sensitivity	86%
NPV	84%
PPV	84%
Correlation	84%

of the 6 sera with "false negative" results in the medac IgG ELISA, 5 were negative and borderline, respectively, when measured with reference II. Considering these measurements, a corrected sensitivity of 98% was calculated.

	IgA	Refe		
√		-	+	
LISA	-	39	13	
MP EI	+	1	20	
\geq				73

Specificity	98%
Sensitivity	61%
NPV	75%
PPV	95%
Correlation	81%

Of the 13 sera with "false negative" results in the medac IgA ELISA, 9 were negative and borderline, respectively, when measured with reference II. Considering these measurements, a corrected sensitivity of 88% was calculated.

	IgM	Refe		
		-	+	
ISA	-	40	8	
MP ELISA	+	0	31	
\subseteq				79

Specificity	100%
Sensitivity	80%
NPV	83%
PPV	100%
Correlation	90%

Of the 8 sera with "false negative" results in the medac IgM ELISA, all were clearly negative when measured with reference II. Considering these measurements, a corrected sensitivity of 100% was calculated.

Specific prevalence


To date, the frequency of *M. pneumoniae* infections as well as their serological reflection has been discussed with controversy. To get a current insight, results obtained in sera from three cohorts were compared in MP ELISA and C-EIA: blood donors (general prevalence), hospitalised children, and adults with acute respiratory infections.



Analysing the antibody constellation, sick and healthy cohorts may be well differentiated from each other. The high number of borderline IgG and IgA results obtained with the commercialised C-EIA was striking (see table).

		IgG		IgA		IgM	
		medac	C-EIA	medac	C-EIA	medac	C-EIA
Blood donors (n = 300)	+ +/- -	39% 11% 50%	30% 22% 48%	6% 3% 91%	7% 17% 76%	2% 2% 96%	1% 1% 98%
Hospitalised children (n = 166)	+ +/-	67% 4% 29%	63% 10% 27%	9% 3% 88%	13% 16% 71%	28% 5% 67%	35% 5% 60%
Hospitalised adults (n = 189)	+ +/-	44% 5% 51%	29% 14% 57%	10% 3% 87%	13% 3% 84%	21% 5% 74%	25% 1% 74%

The particle agglutination test (PAT) is a screening test for *M. pneumoniae* antibodies. In total, 193 sera from patients with respiratory diseases were measured with PAT (negative: n=31; titres 40-160: n=107; titres 320-640: n=27; titres ≥1280: n=28). The following graph shows the corresponding results obtained with Mycoplasma pneumoniae-IgG-, IgA-, and IgM-ELISA medac.

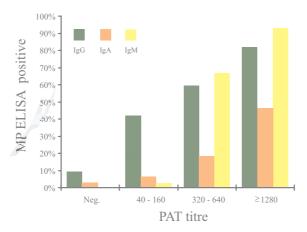



The Mycoplasma pneumoniae ELISAs medac provide differentiated results in regard to IgG, IgA, IgM antibodies. Thus, the course of disease can be concretely assessed. By using PAT, differentiation of the three immunoglobulin isotypes is not possible.


Automation



The automation of Mycoplasma pneumoniae-IgG-, IgA-, and IgM-ELISA medac was verified in various open systems/ELISA processors. The concordance between manual and automated test runs is reflected by the very good correlation between the values obtained (see graphs).

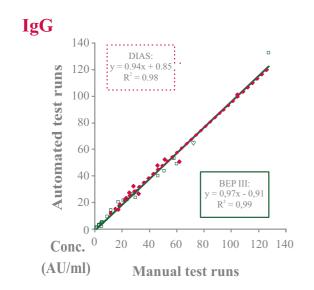

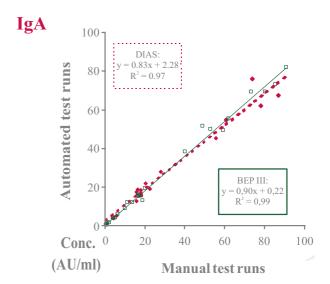

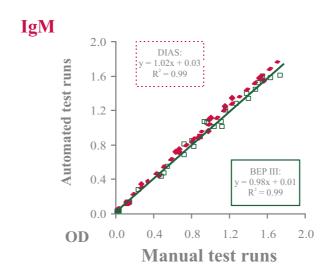

Precision

To establish intra-assay and interassay variance, each five sera of different reactivity were tested both manually and with DSX.



For assessment of intra-assay variance these sera were each tested in a 22-fold assay. For assessment of interassay variance these sera were each tested by 11 independent test procedures.

Intraassay variance

	IgG		IgA		IgM	
Serum	AU/ml	VK	AU/ml	VK	OD	VK
1	6	6%	6	3%	0,021	18%
2	16	4%	12	6%	0,542	3%
3	36	5%	21	3%	0,792	6%
4	50	6%	27	2%	0,990	4%
5	88	5%	116	3%	1,480	4%

Interassay variance

	IgG		IgA		IgM	
Serum	AU/ml	VK	AU/ml	VK	OD	VK
1	3	4%	6	3%	0,033	11%
2	34	5%	21	6%	0,651	2%
3	69	5%	22	5%	0,838	2%
4	82	5%	115	10%	1,141	2%
5	128	7%	126	5%	1,611	2%

Diagnostika

Mycoplasma pneumoniae

References



Clyde WA jr: Clinical overview of typical *Mycoplasma pneumoniae* infections. Clin Infect Dis 17 Suppl. 1, S32-36 (1993).

Daxboeck F, Kircher K, Krause R, Heinzl H, Wenisch C, Stanek G: Effect of stage on antibody titer to *Mycoplasma pneumoniae*. Scand J Infect Dis 34, 577-579 (2002).

Dionisio D, Valassina M, Uberti M, Fabbri C, Parri F, Saffi EG: *Mycoplasma pneumoniae* non-pulmonary infection presenting with pharyngitis, polyarthritis and localized exanthem. Scan J Infect Dis 33, 782-783 (2001).

Döller G, Döller PC, Jacobs E, Schuy W: Zur Differentialdiagnostik von Infektionen des Respirationstrakts. Diagnose & Labor 43, 21-33 (1993).

Dorigo-Zetsma JW, Zaat SAJ, Wertheim-van Dillen PME, Spanjaard L, Rijntjes J, van Waveren G, Jensen JS, Angulo AF, Dankert J: Comparison of PCR, culture and serological tests for diagnosis of *Mycoplasma pneumoniae* respiratory tract infection in children. J Clin Microbiol 37, 14-17 (1999).

Drasbek M, Nielsen PK, Persson K, Birkelund S, Christiansen G: Immune response to *Mycoplasma pneumoniae* P1 and P116 in patients with atypical pneumonia analyzed by ELISA. BMC Microbiol 4, 7 ff. (2004).

Duffy MF, Walker ID, Browning GF: The immunoreactive 116 kDA surface protein of *Mycoplasma pneumoniae* is encoded in an operon. Microbiology 143, 3391-3402 (1997).

Ferwerda A, Moll HA, de Groot R: Respiratory tract infections by *Mycoplasma pneumoniae* in children: a review of diagnostic and therapeutic measures. Eur J Pediatr 160, 483-491 (2001).

Hammerschlag, MR: Mycoplasma pneumoniae infections. Curr Opin Infect Dis 14, 181-186 (2001).

Himmelreich R, Hilbert H, Plagens H, Pirkl E, Bi-Chen L, Herrmann R: Complete sequence analysis of the genome of the bacterium *Mycoplasma pneumoniae*. Nucleic Acids Res 24, 4420-4449 (1996).

Jacobs E: Das Adhäsin von *Mycoplasma pneumoniae*: Seine Bedeutung als Virulenzfaktor in der Pathogenese und in der Diagnostik. Klin Lab 40, 228-229 (1994).

Jacobs E: Serological Diagnosis of *Mycoplasma pneumoniae* infections: A critical review of current procedures. Clin Infect Dis 17 Suppl. 1, S79-82 (1993).

Kleemola M, Käyhty H: Increase in titers of antibodies to *Mycoplasma pneumoniae* in patients with purulent meningitis. J Infect Dis 146, 284-288 (1982).

Krause D: *Mycoplasma pneumoniae* cytadherence: organization and assembly of the attachment organelle. Trends Microbiol 6, 15-18 (1998).

Krause D: Mycoplasma pneumoniae cytadherence: unravelling the tie that binds. Mol Microbiol 20, 247-253 (1996).

Layh-Schmitt G, Podtelejnikow A, Mann M: Proteins complexed to the P1 adhesin of *Mycoplasma pneumoniae*. Microbiology 146, 741-747 (2000).

Rosengarten R: Infektionen durch Mykoplasmen: Neue Aspekte zur Erregerbiologie und Pathogenese. Antibiotika Monitor 1, 3-12 (1998).

Sotgiu S, Pugliatti M, Rosati G, Deiana GA, Sechi GP: Neurological disorders associated with *Mycoplasma pneumo-niae* infection. Eur J Neurol 10, 165-168 (2003).

References

Thacker WL, Talkington DF: Analysis of complement fixation and commercial enzyme immunoassays for detection of antibodies to *Mycoplasma pneumoniae* in human serum. Clin Diagn Lab Immunol 7,778-780 (2000).



Waris ME, Toikka P, Saarinen T, Nikkari S, Meurman O, Vainiopää R, Mertsola J, Ruuskanen O: Diagnosis of *Mycoplasma pneumoniae* pneumonia in children. J Clin Microbiol 36, 3155-3159 (1998).

Watkins-Riedel T, Stanek G, Daxboeck F: Comparison of SeroMP IgA with four other commercially assays for serodiagnosis of *Mycoplasma pneumoniae* pneumonia. Diagn Microbiol Infect Dis 40, 21-25 (2001).

Dorigo-Zetsma JW, Wilbrink B, Dankert J, Zaat SAJ: *Mycoplasma pneumoniae* P1 Type 1- and Type 2-specific sequences within the P1 cytadhesin gene of individual strains. Infect Immun 96, 5612-5618 (2001).

Austria: medac

Dr. Maria Kleindel

Tel.: ++43 2773 43574 Fax: ++43 2773 43574

e-mail: m.kleindel@medac.de

Belgium medac

Mr. Georges Wauthier and Luxembourg: Tel.: ++32 81 74 84 83

++3281748483 Fax:

g.wauthier@medac.de E-mail:

Czech medac

Republic Dr. Anna Manthey

and Tel.: ++42 02 41 40 26 18 Slovac ++42 02 41 40 26 18 Fax:

Republic: a.manthey@medac.de e-mail:

For all other countries please contact medac Diagnostic Division, Gemany

Dr. Karen Dreesbach Mrs. Birgit Behnke Mrs. Cornelia Appelius

++49 4103 8006-0 Tel.: Fax: ++49 4103 8006 -359

 $\underline{k.dreesbach@medac.de}$ e-mail:

b.behnke@medac.de

c.appelius@medac.de

Homepage: www.medac.de

Date of issue: June 2004