IVD DATA SHEET

ZAP-70

Concentrated Rabbit Monoclonal Antibody

Intended Use:

For in Vitro Diagnostic Use

Epitomics' Rabbit Monoclonal Anti-Human ZAP-70, Clone EP52, is intended for use to qualitatively identify ZAP-70 by light microscopy in sections of formalin-fixed, paraffin-embedded tissue using immunohistochemical detection methodology. Interpretation of any positive or negative staining must be complemented with the evaluation of proper controls and must be made within the context of the patient's clinical history and other diagnostic tests. Evaluation must be performed by a qualified pathologist.

Catalog number	Description	Dilution
AC-0050A	0.1 ml, concentrated	1:100-1:200
AC-0050B	0.5 ml, concentrated	1:100-1:200
AC-0050	1 ml, concentrated	1:100-1:200
AC-0050BULK	2 ml or more, concentrated	1:100-1:200

Immunogen: A synthetic peptide corresponding to residues of human ZAP-70 protein

Source: Rabbit Monoclonal Antibody

Clone ID: EP52

Isotype: Rabbit IgG

Application: Immunohistochemistry for formalin-fixed paraffin-embedded tissue

Summary and Explanation:

ZAP-70, a Syk-family protein tyrosine kinase, plays a critical role in mediating T cell signal transduction in response to T cell antigen receptor (TCR) activation. It is primarily expressed in T cells and natural killer (NK) cells. Antibody to ZAP-70 also labels mast cells, basophils and pro/pre B cells but not mature B cells. ZAP-70 antibody is useful in identification of the subtype of chronic lymphocytic leukemia (CLL). Anti-ZAP-70 is positive in CLL with mutation of the immunoglobulin heavy-chain variable region (IgVH) genes, but negative in CLL without IgVH mutation. ZAP-70 expression is associated with disease progression in CLL.

Reagent Provided:

Antibody to ZAP-70 is affinity purified and diluted in 10 mM phosphate buffered saline (PBS), pH 7.2 containing 1% bovine serum albumin (BSA) and 0.09% sodium azide (NaN₃).

Storage and Stability:

Store at 2-8 °C. Don't use after expiration date provided on the vial. End user must validate any storage conditions other than those specified.

Procedures Recommended:

1. Pretreatment: Epitope retrieval using citrate buffer (catalog #: SP-0001) with a pressure cooker.

2. Endogenous peroxidase block: Block for 10 minutes at room temperature using peroxidase solution (catalog #: SP-0002).

3. Protein block: Block for 10 minutes at room temperature using blocking solution (catalog #: SP-0003).

4. Primary antibody: Incubate for 30 minutes.

5. Detection: Follow instructions from the selected detection system (EpiPrecision™, a Biotin Streptavidin-HRP Detection, catalog #: DK-0001, 0003, or EpiVision™, a Rabbit Polymer Detection, catalog # DK-0002, 0004).

The antibody dilution and protocol may vary depending on the specimen preparation and specific application. Optimal conditions should be determined by the individual laboratory.

Performance Characteristics:

This antibody gives cytoplasmic/nuclear staining in positive cells. The recommended positive controls are tonsil for normal tissue and chronic lymphocytic leukemia for abnormal tissue.

Limitations:

Immunohistochemistry is a complex process. Variation in tissue selection, tissue processing, antigen retrieval, peroxidase activity, detection systems and improper counterstaining may cause variation in results.

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Epitomics, Inc.
MDSS GmbH